


Test Instructions

Please use test as directed to ensure the most reliable results.

Testing Kit Includes: (1) Urine Collection Cup, (1) Tube of Buffer Solution, (1) Dropper, (1) Test Cassette

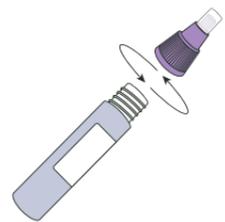


1

Unscrew purple cap from sample tube.

Tube is prefilled with a solution.

Do not remove solution from tube.

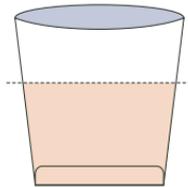


2

Collect urine sample.

Wash your hands.

Urinate in toilet for 2-3 seconds. Then fill cup about halfway.


Do not touch inside of cup.

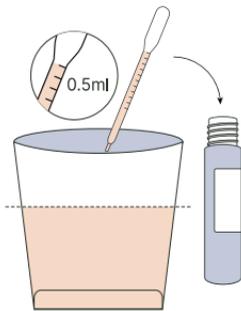

3

Add 0.5ml urine to sample tube.


Use dropper to move urine from cup to sample tube.

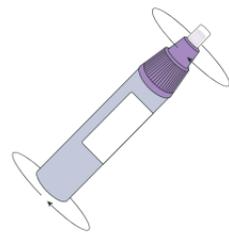

4

Replace purple cap. Swirl tube for 5 seconds.


Gently swirl for 5 seconds to mix urine with solution.

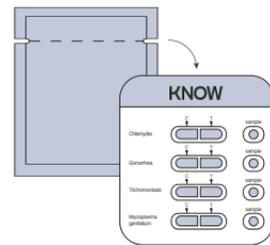

5

Prepare test cassette.


Remove test cassette from foil pouch and place on a flat surface.

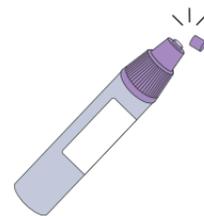

6

Remove clear cap. Snap off purple tip from sample tube cap.

Gently break to avoid spilling sample.

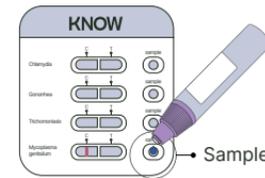

7

Carefully apply 3 drops to each 'Sample' region of the cassette.

Too many (5+) drops can produce invalid results.



8



Wait 15 minutes.

Do not disturb test cassette.

Wait no longer than 30 minutes to read results and take a picture of your cassette.

How to Read Results



POSITIVE

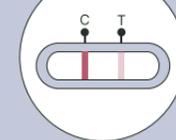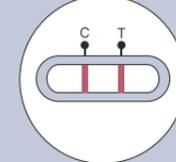

Noticeable T-line

The presence of a noticeable T-line indicates a positive result.



NEGATIVE

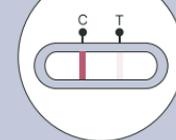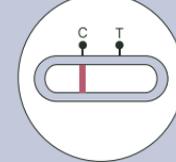

Barely Visible T-line

The absence of a noticeable T-line indicates a negative result. Contact support@testwithknow.com if you have questions about your results.*

INVALID*

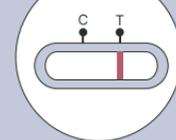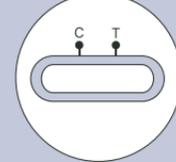

If neither line appears, or if only the T-line appears, your test is considered invalid.*



If a test is positive, seek medical treatment.

Scan to purchase a Telehealth appointment, and connect with a board-certified physician.

*Take a photo of the cassette and email to support@testwithknow.com. Photo must be taken within 30 minutes of testing.



testwithknow.com

The Know Rapid STD test is a urine test only intended to screen for the presence of the sexually transmitted infections listed below. It is not meant for any other purpose and does not test for any other conditions.

Chlamydia

Gonorrhea

Trichomoniasis

Mycoplasma genitalium

Know kit is only for one-time use. If for any reason you cannot complete the test as described on this package, you must restart the process with an unused kit.



testwithknow.com/std-test/positive-result

If you tested positive, or tested negative but still have symptoms, scan to purchase a Telehealth appointment and connect with a board-certified physician.

KNOW.

KNOW.



You can take a *deep breath*.
You'll know within minutes.

INSTRUCTIONS FOR TESTING